

Volume 5

The Tech Bulletin is designed to quickly and easily provide useful information regarding Otto Bock Products and Services. This issue will discuss how to remove the protective cover from the Otto Bock Compact® to allow you to connect the Compact® to your PC.

When connecting the Compact® to your computer using the fiber optic programming cable follow the steps outlined below:

- 1) Carefully remove the protective elastomer cap. Grasp the protective elastomer cap between the thumb and fingers and pull the plastic tongues at the lower end of the elastomer cap out of the recesses on both sides of the knee (Figure 1).



Figure 1



Figure 2

- 2) Insert the plug of the PC interface cable into the PC programming port of the Compact® and establish the connection to the PC (Figure 2).
- 3) After finishing the programming, the elastomer protective cap *must* be reinstalled on the Compact. This is done by reinserting the tongues of the cap back into the recesses on each side of the knee.

Note: Programming the Compact® requires that either Sliders 3.3 or 3.2 be installed on the computer. This is the same software you use for the C-Leg®.