
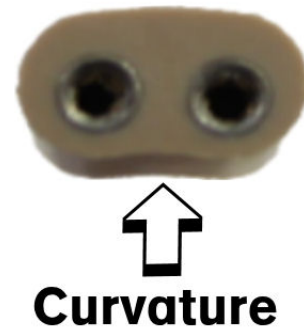


March 2007

The Tech Bulletin is designed to quickly and easily provide useful information regarding Otto Bock Products and Services. In this issue, we will focus on the Pediatric System 2000 Battery Cable Attachment to the 4-in-1 Controllers.

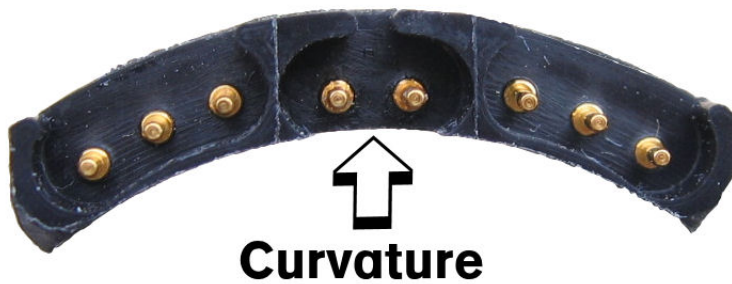

To avoid blowing a fuse in each Battery Pack inserted into the receptacle, match the slight curvature of the Plug to the curvature on the Controller. This is to be done when installing the Battery Cable into the 9E169 or 9E170 4-in-1 Controller.

Please contact our Product Support Specialty Team @ 1-800-328-4058 with any questions.