

## May 2008

The Tech Bulletin is designed to quickly and easily provide useful information regarding Otto Bock products and services. In this issue, we will focus on pairing a new remote control (4X150) with an existing C-Leg®.

When replacing an existing 4X150 remote for the C-Leg it will need to be paired with the knee. To accomplish this task a paper clip works well.



Given that only one joint should be paired with a remote control, it must be ensured that no other joint is within 3 meters during the pairing.

Looking at the face of the remote, you will notice there is a small hole between buttons 1 and 2. Holding the remote at least 12 to 18 inches away and no more than 3 feet away from the knee, insert the paper clip into the hole and you will feel a button. Apply slight pressure and hold button down until you hear 5 beeps indicating that the remote is now paired to the knee.

Please contact our Product Support Specialty Team at 800.328.4058 with any questions.

[Download](#) a copy of this tech tip or our other tips.