


## Transfer Settings Between Patient's C-Leg and Loaner C-Leg

C-Soft v1.2 & C-Soft v1.0

### Save settings from patient's C-Leg to the computer


- 1) Connect BionicLink or C-Leg cable to the patient's knee
- 2) Start C-Soft
- 3) Navigate to the Dataoverview screen

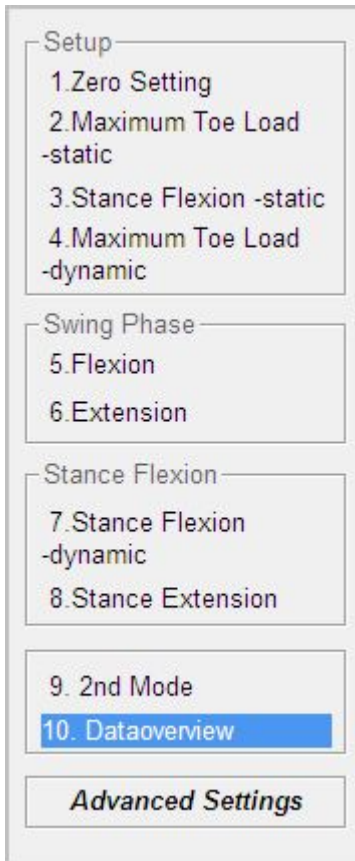

- 4) Click the button with the picture of the C-Leg to file cabinet.



- 5) Give the job a name. Once the job is saved, it will appear under the patient's name in the patient tree.

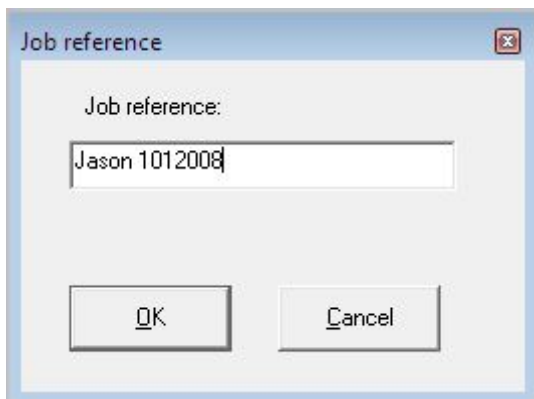

- 6) Close C-Soft

## Transfer Settings Between Patient's C-Leg and Loaner C-Leg

*C-Soft v1.2 & C-Soft v1.0*

### Transfer settings from computer to loaner C-Leg

- 1) Connect BionicLink or C-Leg cable to the loaner knee
- 2) Right click the job and select "Edit job..." to start C-Soft.



- 3) Navigate to the Dataoverview screen
- 4) Click the button with the picture of a file cabinet to C-Leg to transfer the settings from the computer to the C-Leg



- 5) Close C-Soft