

Installing Ottobock Drivers from Ottobock Website

This guide can be used when installing 60X5/60X1 drivers on Windows 7 or Windows XP.

Step 1: Download drivers from Ottobock website

Click the following link and save the drivers to your desktop:


http://academy.ottobockus.com/software/Drivers/USB_DRIVERS_2.0.exe

After saving the drivers to your desktop, double click the **USB_DRIVERS_2.0** file to unzip the drivers.




USB_DRIVE
RS_2.0.exe

Click the **Extract** button.

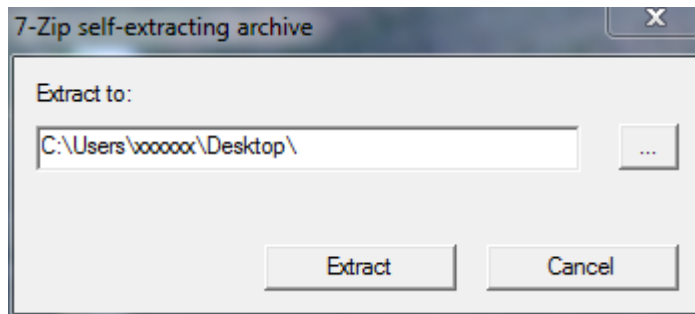


After clicking Extract, a new **USB DRIVERS 2.0** folder will be created on the desktop.

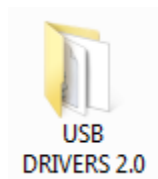


Step 2: Install Drivers

- Plug the 60X5/60X1 adapter into the computer.
- Open the **USB DRIVERS 2.0** folder on your desktop.
- Double click the **Setup.exe** file to begin the installation.
- Click **Next** on the welcome screen.

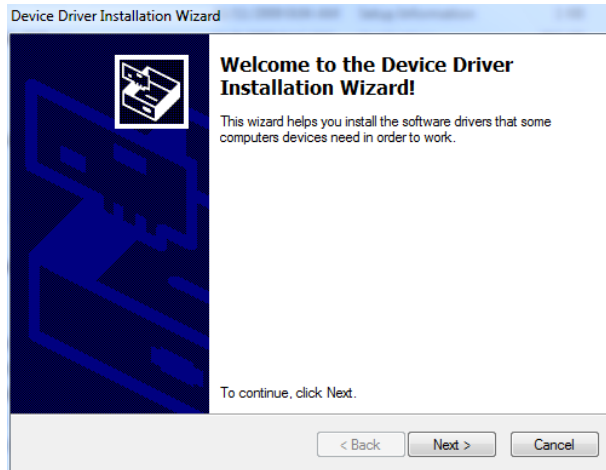

- During the installation you may be prompted several times with a security message similar to the following screenshots. If you are prompted with this message, click **Install this driver software anyway** on Windows 7 or **Continue Anyway** on Windows XP.

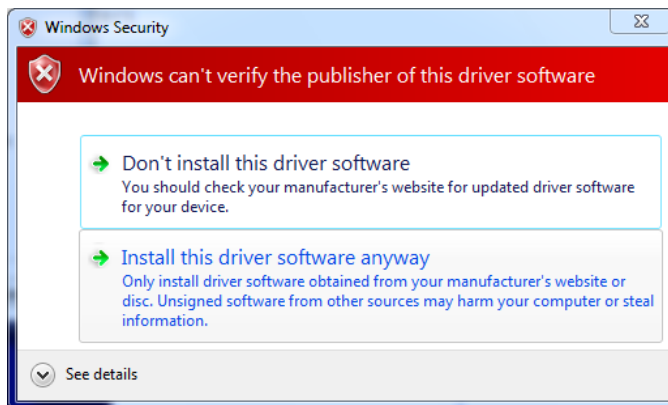

Windows 7



Windows XP

- Click Finish. The 60X5/60X1 drivers are installed.

