



QUALITY FOR LIFE

Creating a Desktop Shortcut to the OBSS 6.6 Directory

This guide can be used when creating a shortcut to the Otto Bock Shape System default directory. The default directory is used to store .bd4 files and OBSS order forms.

Creating a Desktop Shortcut

- Right click on the desktop
- Click 'New'
- Click 'Shortcut'
- Next to 'Type the location of the item' field click the 'Browse...' button

Follow the instructions below for your version of Windows.

Windows XP

- Under Desktop highlight the following directory:
My Computer\C:\Documents and Settings\All Users\Shared Documents\Otto Bock\Shape6
- Click 'OK'
- Click 'Next'
- Leave the shortcut name as 'Shape6' and click 'Finish'

Windows Vista/7

- Under Desktop highlight Libraries\Documents\Public Documents\Otto Bock\Shape6
- Click 'OK'
- Click 'Next'
- Leave the shortcut name as 'Shape6' and click 'Finish'

If you need additional technical support, contact Otto Bock at 800-328-4058.