




QUALITY FOR LIFE

Installing Patriot Drivers on Windows 7



This guide can be used when installing Polhemus Patriot Drivers on a Windows 7 computer. Windows 7 will not install Patriot driver software automatically.

Installing Patriot drivers on Windows 7


- Plug the Patriot system into a USB port
- Turn the Patriot 'ON'
- Open the Device Manager
 - Click the Start button 
 - In *Search programs and files* type 'device manager' and hit the Enter key

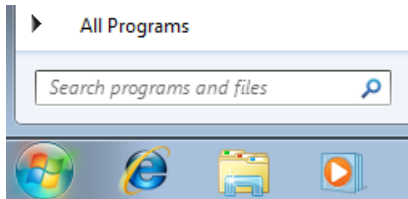

- Insert the OBSS 6.6 software CD into the computer
 - If the AutoPlay window opens or the OBSS software installation begins, close the AutoPlay window and cancel the installation.
- In the Device Manager navigate to and open *Other devices*
 - The Patriot system will be located under *Other devices* as an *Unknown device*. Double left click *Unknown device*.
 - Click the *Update Driver* button
 - Click on *Browse my computer for driver software*
 - In the browse window, click the browse button and navigate to your CD drive. Highlight (left click) the CD drive and click OK.

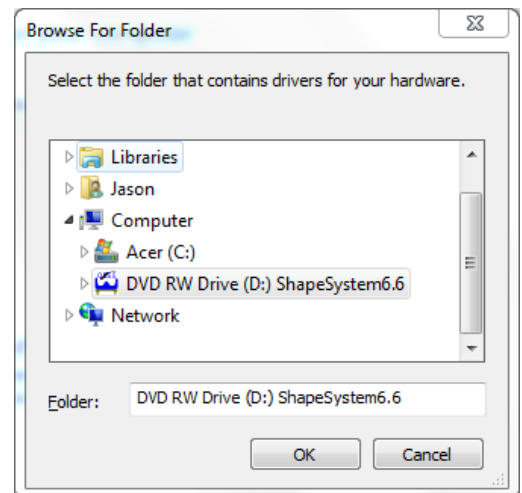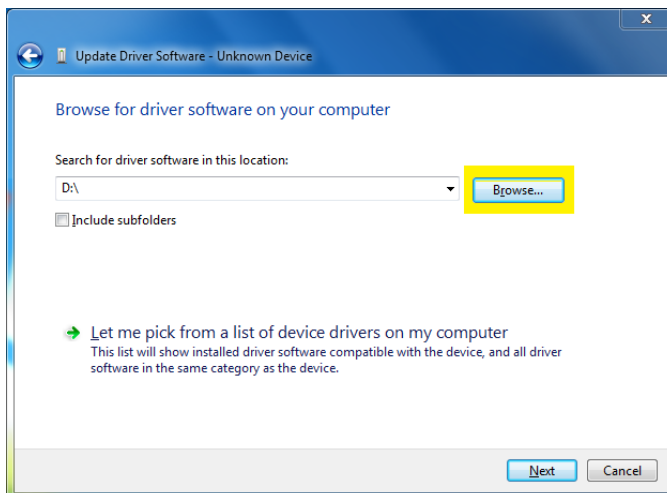

- Click the Next button
- If a Windows Security message is displayed, click Install.

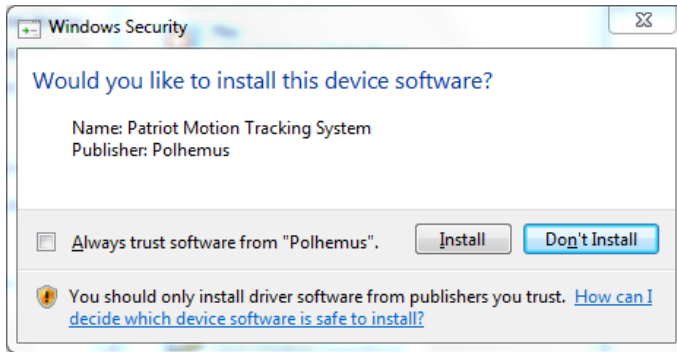

- When the driver is finished installing click the Close button.
- Click Close again and Close out of the Device Manager
- The Patriot driver should now be installed.

If you need additional technical support, contact Otto Bock at 800-328-4058.