




QUALITY FOR LIFE

Installing Patriot Drivers on Windows Vista


This guide can be used when installing Polhemus Patriot drivers on a Windows Vista computer. Windows Vista will not install Patriot driver software automatically.

Note: During the installation process you will be prompted by the User Account Control (UAC) for permission to continue the installation. Click 'Continue' when these prompts are shown.


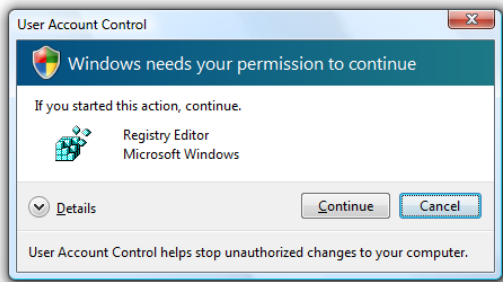


Installing Patriot drivers on Windows Vista


- Plug the Patriot system into a USB port
- Turn the Patriot 'ON'
- If the Found New Hardware wizard opens click *Cancel*.
- Open the Device Manager
 - Click the Start button 
 - In *Start Search* type 'device manager' and hit the Enter key

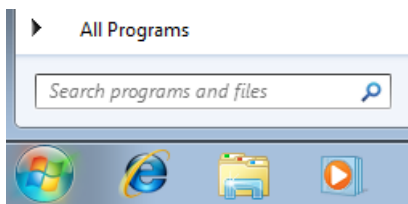

- Insert the OBSS 6.6 software CD into the computer
 - If the AutoPlay window opens or the OBSS software installation begins, close the AutoPlay window and cancel the installation.
- In the Device Manager navigate to and open *Other devices*
 - The Patriot system will be located under *Other devices* as an *Unknown device*. Double left click *Unknown device*.
 - Click the *Reinstall Driver...* button
 - Click on *Browse my computer for driver software*
 - In the browse window, click the browse button and navigate to your CD drive. Highlight (left click) the CD drive and click OK.

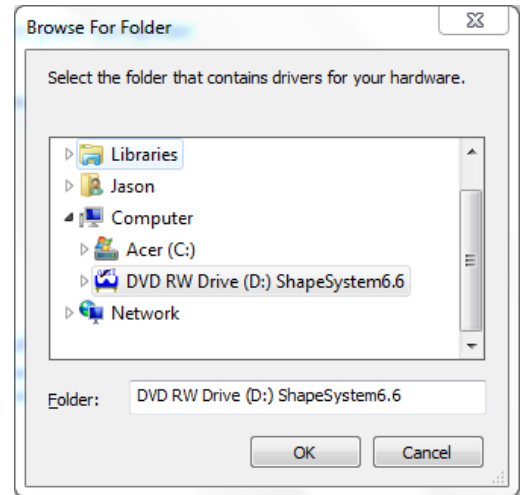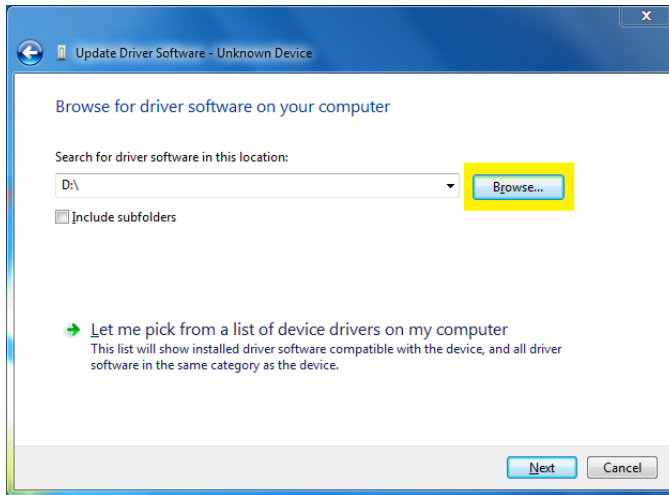

- Click the Next button
- If a Windows Security message is displayed, click Install.

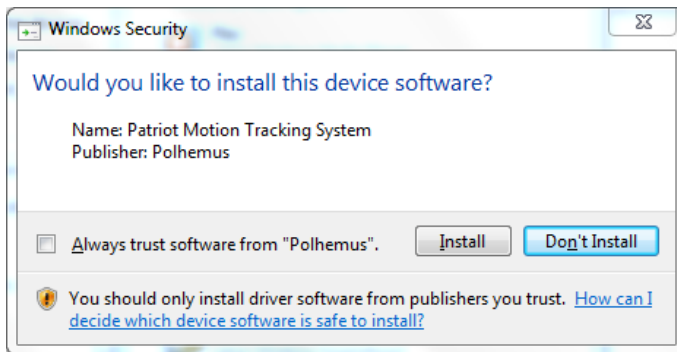

- When the driver is finished installing click the Close button.
- Click Close again and Close out of the Device Manager
- The Patriot driver should now be installed.

If you need additional technical support, contact Otto Bock at 800-328-4058.