



QUALITY FOR LIFE

Installing Patriot Drivers on Windows XP

This guide can be used when installing Polhemus Patriot drivers on a Windows XP computer.

Installing Patriot drivers on Windows XP

- Plug the Patriot system into a USB port
- Insert the OBSS 6.6 software CD into the computer
 - If the AutoPlay window opens or the OBSS software installation begins, close the AutoPlay window and cancel the installation.
- Turn the Patriot 'ON'
- The *Found New Hardware Wizard* should open
 - If prompted to connect to Windows Update to search for software, select *No, not this time* and click the Next button.
 - Select *Install the software automatically (Recommended)* and click Next.
 - When the wizard is complete, click Finish.
 - The *Found New Hardware Wizard* will start again. Repeat the above steps and select Finish.
 - The Patriot driver should now be installed.

If you need additional technical support, contact Otto Bock at 800-328-4058.