



QUALITY FOR LIFE

Otto Bock Shape System 6.6 Installation Guide

This guide can be used when installing Otto Bock Shape System (OBSS) on a Windows based computer. OBSS 6.6 requires Windows XP SP3, Vista or Windows 7.

Installing OBSS 6.6

Note: You must have administrator rights to install the software.

To run the installation program, start Windows and insert the OBSS 6.6 CD into the CD drive. The setup program should run automatically when the CD is inserted into the CD drive. If this is not the case, find your CD drive under 'My Computer' and start the installation program by left double clicking 'setup.exe.'

Tip - Windows 7/Vista: Click 'Yes' when prompted with the following message: Do you want to allow the following program to make changes to this computer?

Screen by Screen Installation

- Select the language – Select 'English' and click OK.
- Welcome to the InstallShield Wizard - Click Next.
- License Agreement – Accept the license agreement and click Next
- Customer Information – Enter your Name and Organization and click Next
- Select Country – Select the 'United States of America' and click Next
- Setup Type – Select 'Complete' and click Next
- Ready to Install the Program – Click Install
- InstallShield Wizard Completed – Click Finish

Start the Otto Bock Shape System 6.6 by clicking the Otto Bock Data Station icon on your desktop.

If you need additional technical support, contact Otto Bock at 800-328-4058.